Helping hand Nicky

The helping hand Nicky is designed for daily assistance to grab or pick up small objects on the floor. Avoid picking up heavy objects as this might cause damage to your helping hand Nicky.

Hold the helping hand Nicky firmly in your left or right hand with your index and middle finger around the trigger. Point towards the object you want to pick up and pull the trigger. The clamp will close around the object you want to pick up. Keep the trigger firmly pulled until you want to release your object. Avoid using brute pulling force as this might cause damage to your helping hand Nicky.

The front of the helping hand Nicky is equipped with a small magnet. It can help you to pick up small magnetic object such as keys, coins or cans.

! Avoid using it on magnetic-sensitive objects such as bankcards as it might cause damage.

! Avoid picking up heavy objects as it might cause damage.

The front of the helping hand Nicky is equipped with a clothing picker. It will help you to easily pick up clothes or socks on the floor.

The helping hand Nicky can be cleaned with normal household cleaning products. Avoid using aggressive cleaning products which can cause discoloration.

Subject to technical alterations. © 2016

VERMEIREN

Opening up new horizons in mobility

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Vermeiren Nederland B.V.