



Material			
No.	Plan	Article	Quantity
4	YP05996	27916	2
5	XP01851	2993570	1
6	XP01439	2993524	10
7	P00030394	2995366	4
8	X002136	1992414	4
9	M6	2994059	2
10	M8x115	2994008	1
11	NUTM8	2994009	1
12	M6 DIN125	2994068	2
13	ScrewD3.9X9.5	2994075	4
14	M6x45DIN912	2994058	2

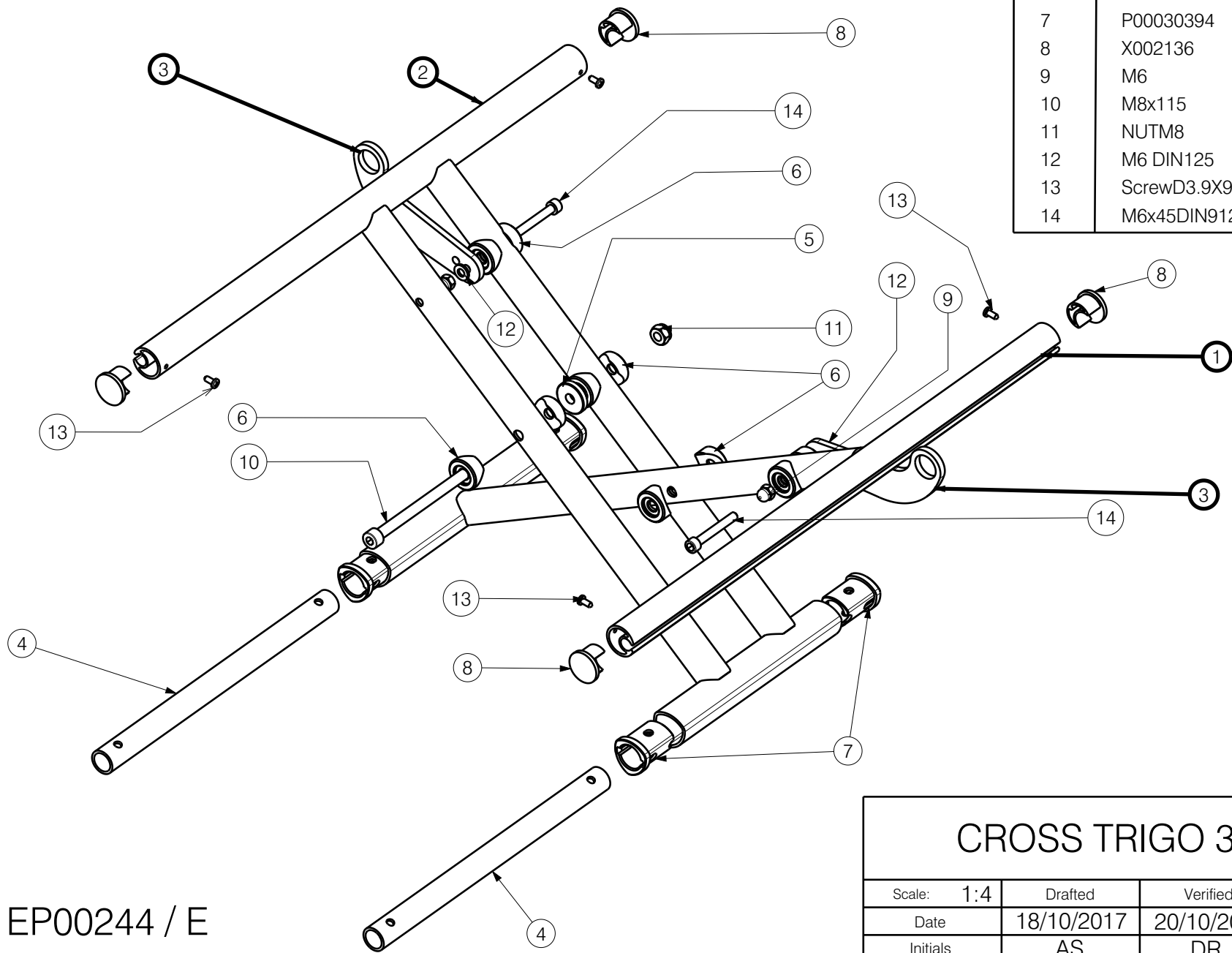

EP00244 / E

<h1>CROSS TRIGO 36-46</h1>				Plan No.
				EP00244 / E
Scale: 1:4	Drafted	Verified	Confirmed	
Date	18/10/2017	20/10/2022	20/10/2022	
Initials	AS	DR	AS	


Page 1/3

Material				
No.	Plan	Article	Quantity	
Width 36cm				
1	YP06850	2756565 (36cm)	1	
	YP06849	2756465 (38cm)	1	
	YP06848	2756365 (40cm)	1	
	YP06847	2756265 (42cm)	1	
	YP06846	2756165 (44cm)	1	
	YP06845	2756065 (46cm)	1	
	YP06844	2755965 (48cm)	1	
2	YP05650	2720265 (50cm)	1	
	YP06843	2755865 (36cm)	1	
	YP06842	2755765 (38cm)	1	
	YP06841	2755665 (40cm)	1	
	YP06840	2755565 (42cm)	1	
	YP06839	2755465 (44cm)	1	
	YP06838	2755365 (46cm)	1	
3	YP06837	2755265 (48cm)	1	
	YP05658	2721065 (50cm)	1	
	YP05691	2719865	2	
Width 38cm				
1	YP06836	2754465 (36cm)	1	
	YP06835	2754365 (38cm)	1	
	YP06834	2754265 (40cm)	1	
	YP06833	2754165 (42cm)	1	
	YP06832	2754065 (44cm)	1	
	YP06831	2753965 (46cm)	1	
	YP06830	2753865 (48cm)	1	
	YP05651	2720365 (50cm)	1	
	2	YP06829	2755165 (36cm)	1
		YP06828	2755065 (38cm)	1
YP06827		2754965 (40cm)	1	
YP06826		2754865 (42cm)	1	
YP06825		2754765 (44cm)	1	
YP06824		2754665 (46cm)	1	
YP06823		2754565 (48cm)	1	
3	YP05659	2721165 (50cm)	1	
	YP05691	2719865	2	

Material				
No.	Plan	Article	Quantity	
Width 40cm				
1	YP06757	2753665 (36cm)	1	
	YP06756	2753565 (38cm)	1	
	YP06755	2753465 (40cm)	1	
	YP06754	2753365 (42cm)	1	
	YP06753	2753265 (44cm)	1	
	YP06752	2753165 (46cm)	1	
	YP06751	2753065 (48cm)	1	
	YP05652	2720465 (50cm)	1	
	2	YP06750	2752965 (36cm)	1
		YP06749	2752865 (38cm)	1
YP06748		2752765 (40cm)	1	
YP06747		2752665 (42cm)	1	
YP06746		2752565 (44cm)	1	
3	YP06745	2752465 (46cm)	1	
	YP06744	2752365 (48cm)	1	
	YP05660	2721265 (50cm)	1	
	YP05692	2719965	2	
	Width 42cm			
1	YP06743	2751565 (36cm)	1	
	YP06742	2751465 (38cm)	1	
	YP06741	2751365 (40cm)	1	
	YP06740	2751265 (42cm)	1	
	YP06739	2751165 (44cm)	1	
	YP06738	2751065 (46cm)	1	
	YP06737	2750965 (48cm)	1	
	YP05653	2720565 (50cm)	1	
	2	YP06736	2752265 (36cm)	1
		YP06735	2752165 (38cm)	1
YP06734		2752065 (40cm)	1	
YP06733		2751965 (42cm)	1	
YP06732		2751865 (44cm)	1	
3	YP06731	2751765 (46cm)	1	
	YP06730	2751665 (48cm)	1	
	YP05661	2721365 (50cm)	1	
	YP05692	2719965	2	

Material				
No.	Plan	Article	Quantity	
Width 44cm				
1	YP08555	2620165 (36cm)	1	
	YP06729	2741765 (38cm)	1	
	YP06728	2741665 (40cm)	1	
	YP06727	2741565 (42cm)	1	
	YP06726	2741465 (44cm)	1	
	YP06725	2741365 (46cm)	1	
	YP06724	2741265 (48cm)	1	
	YP05653	2720665 (50cm)	1	
	2	YP08556	2620265 (36cm)	1
		YP06723	2742365 (38cm)	1
		YP06722	2742265 (40cm)	1
		YP06721	2742165 (42cm)	1
		YP06720	2742065 (44cm)	1
3	YP06719	2741965 (46cm)	1	
	YP06718	2741865 (48cm)	1	
	YP05662	2721465 (50cm)	1	
	YP05693	2720065	2	

EP00244 / E

<h1>CROSS TRIGO 36-46</h1>				Plan No.
				EP00244 / E
Scale: 1:4	Drafted	Verified	Confirmed	
Date	18/10/2017	20/10/2022	20/10/2022	
Initials	AS	DR	AS	

Material			
No.	Plan	Article	Quantity
Width 46cm			
1	YP08557	2620365 (36cm)	1
	YP08558	2620465 (38cm)	1
	YP06717	2740665 (40cm)	1
	YP06716	2740565 (42cm)	1
	YP06715	2740465 (44cm)	1
	YP06714	2740365 (46cm)	1
	YP06713	2740265 (48cm)	1
	YP05655	2720765 (50cm)	1
2	YP08559	2620565 (36cm)	1
	YP08560	2620665 (38cm)	1
	YP06712	2741165 (40cm)	1
	YP06711	2741065 (42cm)	1
	YP06710	2740965 (44cm)	1
	YP06709	2740865 (46cm)	1
	YP06708	2740765 (48cm)	1
	YP05663	2721565 (50cm)	1
3	YP05693	2720065	2

EP00244 / E

CROSS TRIGO 36-46				Plan No.
Page 3/3				EP00244 / E
Scale: 1:4	Drafted	Verified	Confirmed	
Date	18/10/2017	20/10/2022	20/10/2022	
Initials	AS	DR	AS	